


Dermacol for babies

The area of skin around your baby's stoma is likely to be sensitive and, if not kept clean and dry, can become sore. Dermacol is made from soft material that will wriggle with your baby and has a skin-friendly hydrocolloid to protect even the most delicate skin.


"She's stopped getting rashes, thanks to Dermacol."




How to fit Dermacol on your baby

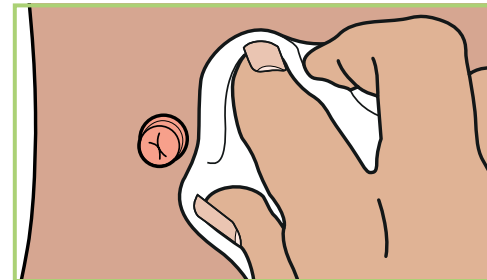


1 Ensure that the skin around the stoma is clean and dry.

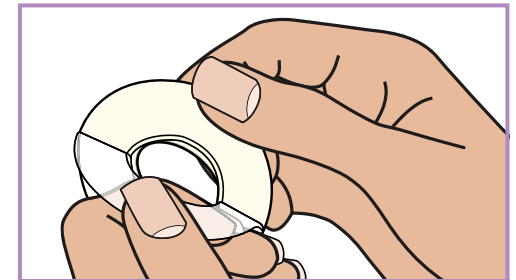

2 Remove film backing.

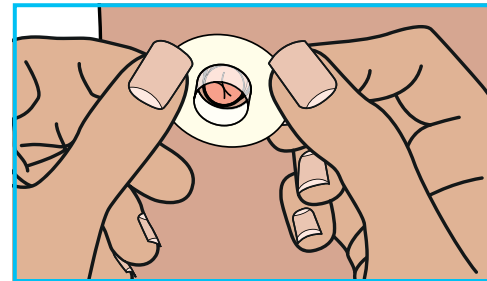


3 Slide collar gently up and around the stoma.

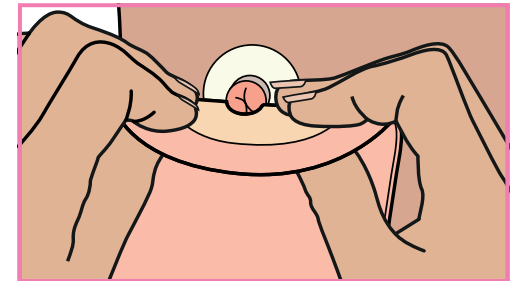

4 Securely connect the pouch or flange to the collar. If collar pushes down the stoma, try method 2 or try cutting the flange slightly larger. Put the pouch on in the usual way.



5 Smooth round with fingers to make sure the pouch is secure. If using a two piece system apply pouch to flange.

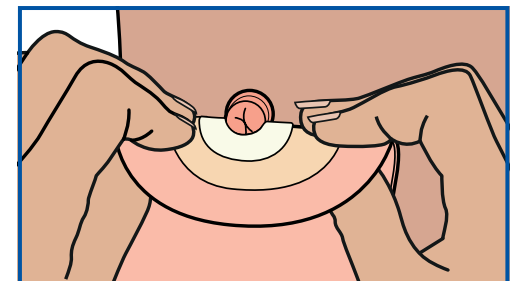

6 Gently remove pouch or flange together with the collar.