



Dermacol for small children

With Dermacol, leaks are one less thing to worry about. The soft collar is designed to bend and flex with even the most active children, providing a leakproof barrier that will keep skin smooth and soft all day long.


"He's a lot happier since we started using Dermacol."

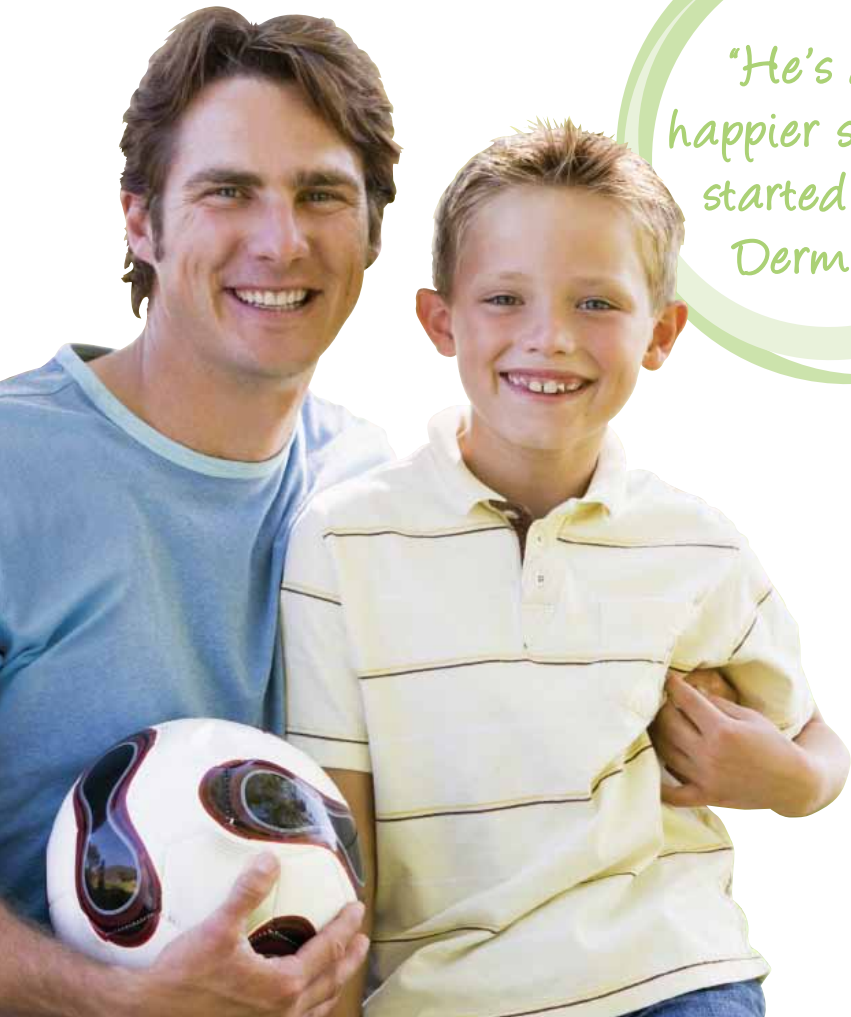


How to fit Dermacol on your child

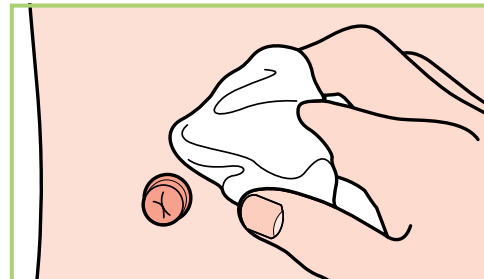


1 Ensure that the skin around the stoma is clean and dry.

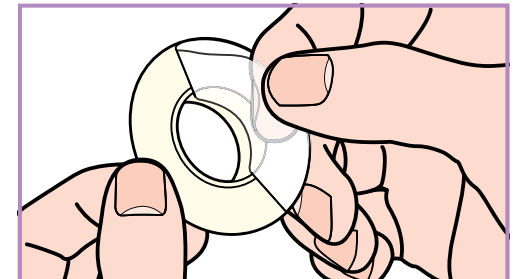


2 Remove film backing.

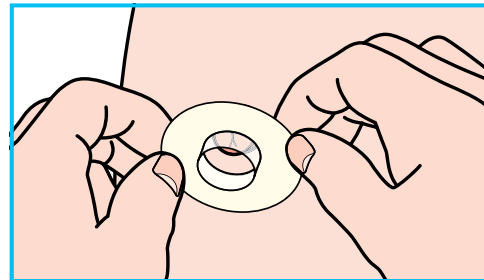


3 Slide collar gently up and around the stoma.

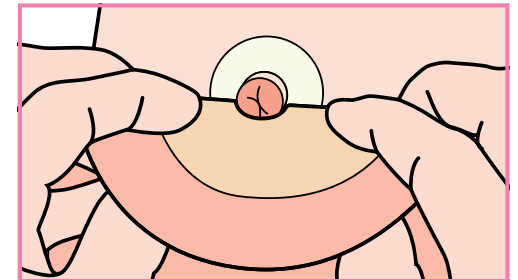

4 Securely connect the pouch or flange to the collar. If collar pushes down the stoma, try method 2 or try cutting the flange slightly larger. Put the pouch on in the usual way.

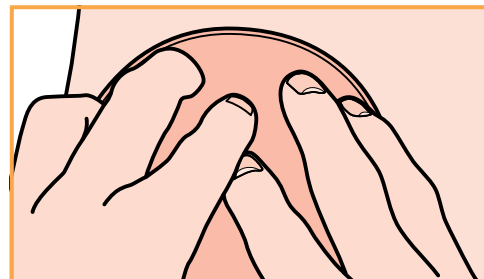

5 Smooth round with fingers to make sure the pouch is secure. If using a two piece system apply pouch to flange.

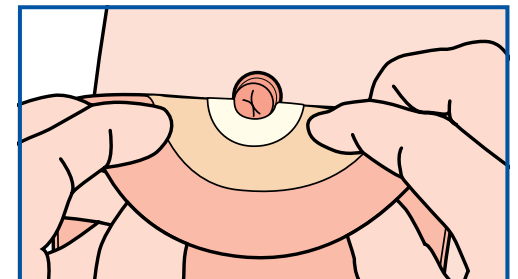

6 Gently remove pouch or flange together with the collar.