


Dermacol for teenagers

As a teenager, your body shape is constantly changing, which can sometimes lead to ill-fitting appliances and leaks. Dermacol has been specifically designed to adapt to these changes and help you to stay leak-free in whatever you do.


"Dermacol fits all shapes, sizes and types of stomas so can help you stay leak-free in whatever you do."

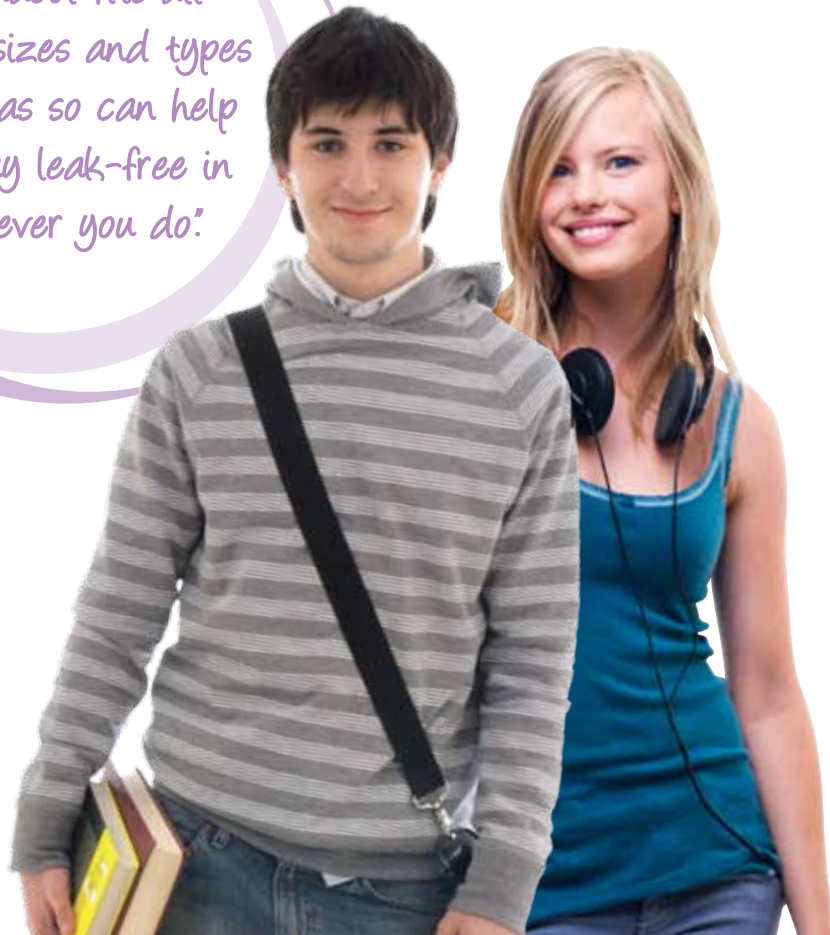


How to fit Dermacol Method 1




1 Ensure that the skin around the stoma is clean and dry.

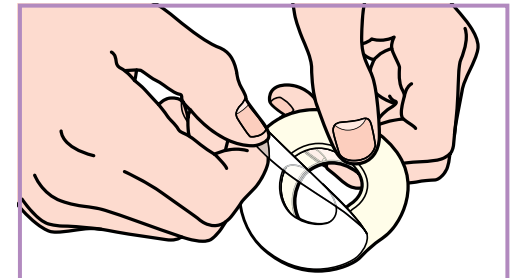

2 Remove film backing.

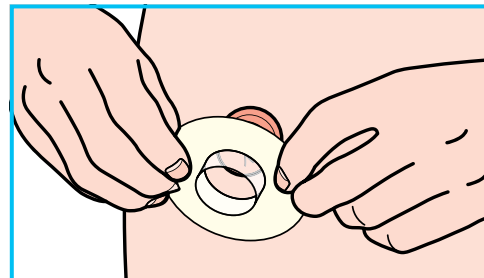


3 Slide collar gently up and around the stoma.

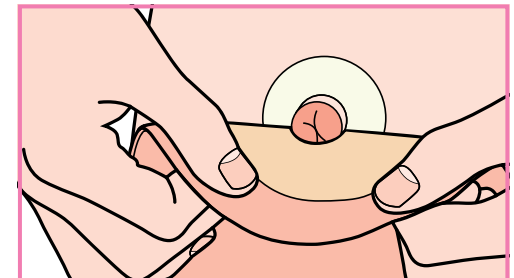

4 Securely connect the pouch or flange to the collar. If collar pushes down the stoma, try method 2 or try cutting the flange slightly larger. Put the pouch on in the usual way.




5 Smooth round with fingers to make sure the pouch is secure. If using a two piece system apply pouch to flange.

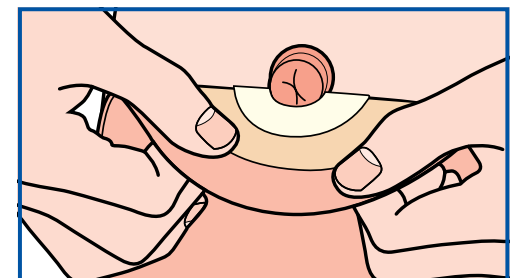


Gently remove pouch or flange together with the collar.


How to fit Dermacol Method 2

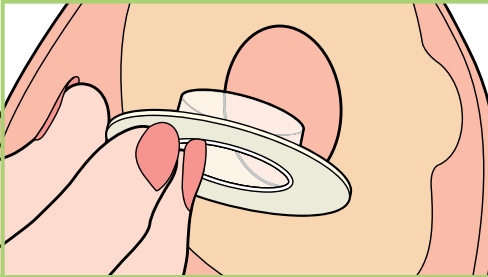


1 Insert the collar into the pouch ensuring the edges of the collar do not get caught.

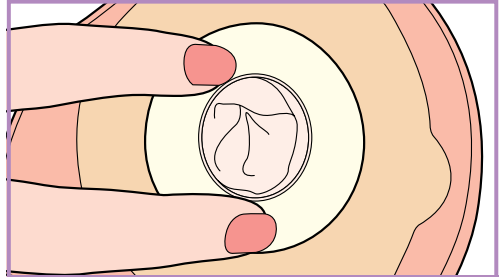


2 Smooth round with the fingers to ensure good adhesion.

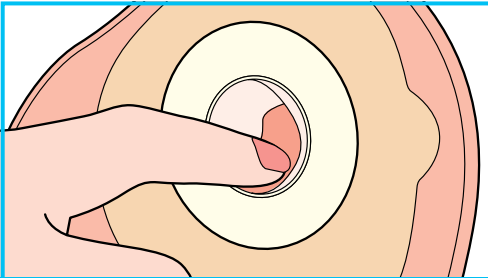


3 Push the collar back through, if required.

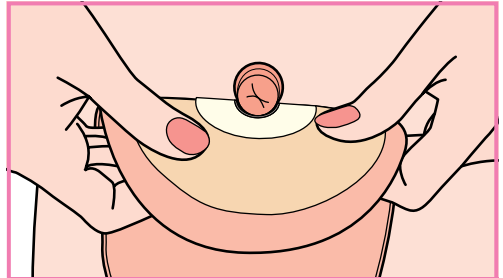

4 Carefully place the stoma through the collar and pouch or flange.

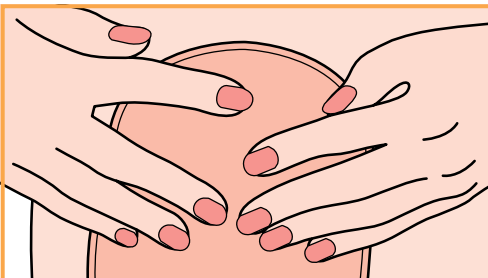

5 The pouch or flange is now safely attached. If using a two-piece system, apply pouch to flange.

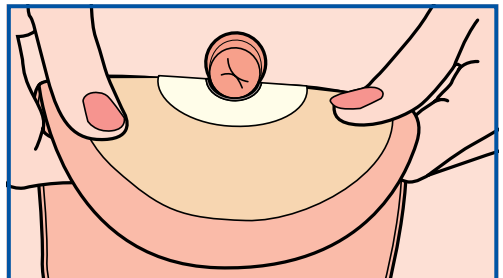

6 Removal
Gently remove pouch or flange together with the collar.